


Using ZOLL OnLine for EKGS/12-Leads

Created: Mar 29, 2021 - 11:23AM by Baumgart, Larry

Using ZOLL OnLine for EKGS/12-Leads

MercyFlight / MercyEMS Crew Chiefs, Base leads, PCR Auditors have access to ZOLL OnLine. Field Staff will have to get with someone to retrieve their information.

1. Log onto your ZOLL OnLine account
2. Select 12-Lead, or CaseReview Basic (this example is CaseReview)

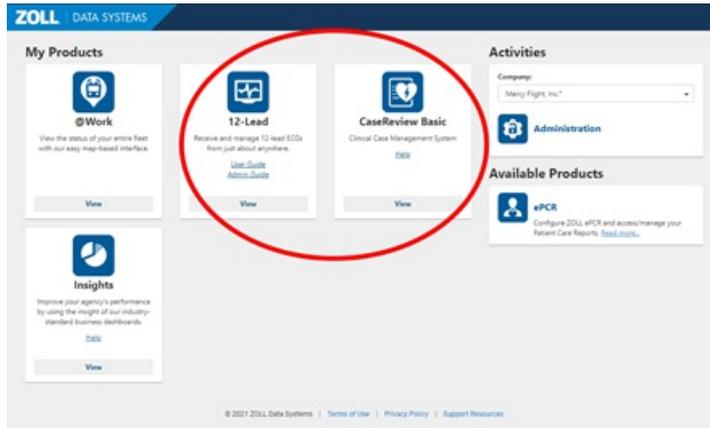

a.


3. Find your uploaded case

a. Click on case

>	3/25/2021 20:12:03	00:29:03	AR12E000936	MEX2	Patient 5177	↓
>	3/25/2021 19:54:14	00:58:33	AR20B046251	MEX6	Patient 1659	↓
>	3/25/2021 19:21:54	01:20:56	AR15L016258	MFX5	Patient 2780	↓
>	3/25/2021 18:51:20	00:30:09	AR15H014766	MFX4	Patient 2769	↓
>	3/25/2021 18:19:06	00:41:23	AR20B046519	MEX13	Patient 1049	↓
>	3/25/2021 16:03:34	00:36:45	AR12E000936	MEX2	Patient 5176	↓
>	3/25/2021 15:38:52	00:38:51	AR12D000750	MEX4	Patient 3462	↓


b.

4. You can playback, or export

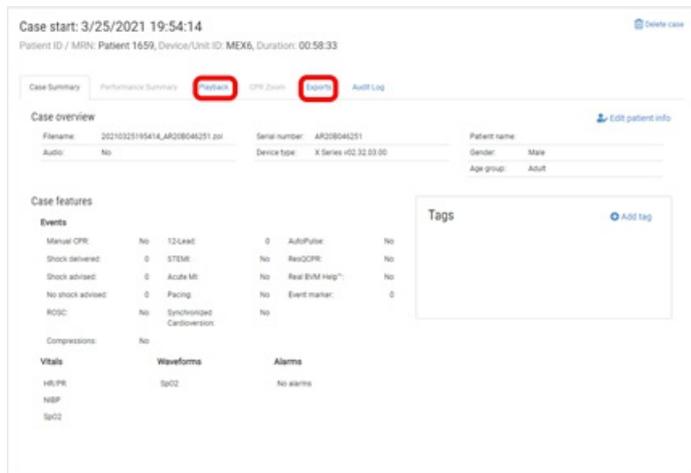

a.

5. Playback lets you browse case

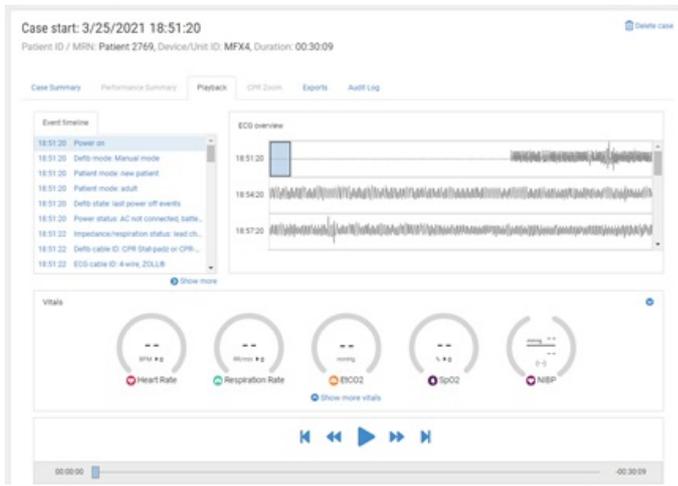

a.


6. You can export a full .PDF to view, or attach the file to your chart

a.

b.

Name	Date modified	Type	Size
Today (1)			
2021-03-29Summary.zip	3/29/2021 10:40 AM	Compressed (zipp...	1,482 KB
Last week (11)			

c.

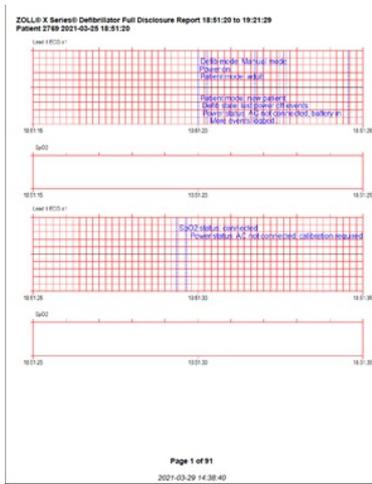

d.

📎 Zoll, EKG, upload, online, casereview, review

📄 Using ZOLL OnLine for EKGs.pdf

Details

Category
Software / Application

Subcategory
ZOLL @Work

Related

INCIDENTS

Crotty, Jennifer

run 21-04934

5446 • Low • Software / Application • emsCharts